sergiogaldani@uol.com.br

PRECISION ELECTROSURGICAL GENERATOR

SMOKE EVACUATOR VS7001

Contains a ULPA Filter with activated carbon. Indicated to eliminate excessive vapor, bacteria and virus. Keeping the surgical environment clean.

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OPTIONAL

Accessories can be selected according to users preference. Consult the accessory catalog.

IMPORTANT NOTES:

The equipment referred meets the requirements of IEC 60601 -1 safety electrical equipment, certified by INMETRO.

IEC 60601 -1-2 Electromagnetic Compatibility.

IEC 60601 -2-2 specific for electrosurgical generators.

ANVISA registered (Brazilian Health Agency).

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THE COMPANY

Deltronix manufactures electrosurgery units and its accessories for medical, dentistry and veterinarian field since 1970. Every equipment is individually tested and calibrated according to national and international standards. Its established success is a result of the finest resources in electric engineering that developed a functional, durable, high quality and easy to use equipment.

ELECTROSURGERY ARE US!

DELTRONIX Equipamentos Ltda.

Barão de Cotegipe Str. 800

Zip Code 14050-420

Ribeirão Preto - SP - Brazil

Phone. +55 (16) 4009-5454 Fax +55 (16) 4009-545

export@deltronix.com.br | eltronix@deltronix.com.l

www.deltronix.com.br

TECHNICAL SPECIFICATIONS:

03 (three) independent and simultaneous Digital Displays to the Cut, Coagulation and Bipolar powers.

12 (twelve) Cut functions (Low, High and Pulsed).

6 (six) Coagulation functions.

Function PPC® (Pulsed Polypectomy Cut/Coag) Pulsed Cut and Coagulation.

Bipolar Function: Precise mode (Microbipolar), Standard mode, MacroBipolar/Bipolar Cut (Bcut®) mode.

Exclusive TRIPOLAR® mode (Monopolar + Bipolar).

3 independent connections being 2 Monopolar and 1 Bipolar, with protection grade IPX7.

Two independent connections for monopolar pencil.

Compatible with Argon Gas Coagulator.

Allows the use of special pencil used on coagulation with argon gas and coagulation /monopolar cut.

FPA® - (FEEDBACK POWERADJUSTMENT®) -Automatic Power Adjustment matching tissue resistance.

Start up self testing function with error code.

Remote control to adjust power of Cut, Coagulation and Bipolar.

Power adjustment with three speeds.

Memory for up to 120 different procedures, individually configurable.

Automatic detection of plate type, single or double.

Audible indication of the function in use, with different tones to CUT, COAG and BIPOLAR.

Stand-By Mode.

Audio adjustment for the function in use in the front panel.

RMPPF® System - Redundant Mechanism for Protection in case of Failure.

CVM® - Consistency Verification Mechanism (firm ware).

Internal Cooling: Natural convection ventilation.

Plate Alarm System: System MRPGraph® (Contact Resistance Monitoring Plate/Patient with Bargraph).

Power adjustment on the control panel, on the digital hand control pencil or on the double footswitch..

REMOTE FUNCTION

Activation and adjustment of Cutting, Coagulation and Bipolar power by automatic remote system.

STAND-BY MODE

Used in situation that requires momentary interruption of the surgical procedures.

SIMULTANEOUS USE FOR 2 PENCILS

Allows the simultaneous and independent action of 2 medical professionals, using both pencil or footswitch.

LIGHTED OUTPUT CONNECTORS (BORNES)

Assist professionals to visualize, specially during videolaparoscopic procedures that demands low lights in the operation room.

FPA® (Feedback Power Adjustment®)

PRECISION Line presents automatic power compensation in accordance with the change in tissue resistance, keeping constant power at all times.

AUTOMATIC DETECTION FOR THE RETURN PLATE

The equipment recognizes the type of plate that is being used releasing the system to use after confirmation of correct connection. Allows using the same cable for both reusable or single use plates.

THE NEW GENERATION OF ELECTROSURGERY

HIGH FREQUENCY ELECTROSURGICAL GENERATORS

The electrosurgical generator Deltronix PRECISION® uses the latest generation of microprocessors. Designed for minor, mid and major complex surgery, specially indicated for Urologic (RTU and EVP), Cardiac and Neurosurgery. The cutting and coagulating in wet field (underwater), allows its use in all kinds of endoscopic resection. The waveform for each function was conveniently set for cardiovascular surgery, being equally tested and approved for mammary resection.

MEMORY

Memory for up to 120 editable and non-volatile different procedures.

RELOAD FUNCTION

Stores immediately the selected values in case of momentary power line failure. All power values selected in any of the used functions are immediately stored, facilitating the return to selected values in case of momentary lack of electricity.

SECURITY SYSTEMS

MRPGraph® System

Visual Monitoring System - for the Patient/Plate contact resistance when using single use double plates; Bargraph indicating the quality of the contact.

RMPPF® System

Redundant Mechanism for Protection in case of CPU Failure.

CVM® System

Consistency Verification Mechanism - Continuous monitoring of all functions.

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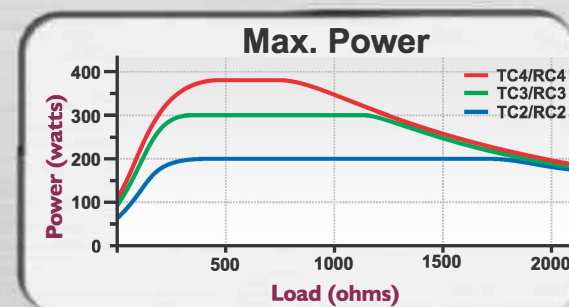
Valid values for declared nominal load, 500 Ohms for Monopolar function and 100 Ohms for Bipolar function. Monopolar modes: the precision power setting is done in 1.0 Watt to 1.0 Watt increments until the end of scale. Bipolar mode: the power setting is done in 0.5 to 0.5 Watt increments until the end of scale.

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Automatic power adjustment with tissue resistance variation.

Note: In the Cut function is suggested to use LOW for well irrigated tissues and HIGH for less irrigated tissues (higher tissue resistance).

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PRECISION ELECTROSURGICAL GENERATOR

GENERAL CHARACTERISTICS:

100-240 V, 50/60 Hz, automatic selected voltage.
Oscillator base frequency: 400kHz.
Built to eliminate shock and equipment damage hazards arising from liquid dripping during usage. IPXI pattern.
Dimension (WxHxL): 325 x 135 x 485 mm.
Net Weight: 6,2 Kg.
Aluminium box with electrostatic paint of high resistance.
Waterproof polycarbonate panel resistant.
Carrying handle (retractable).
Lighted output connectors (bornes).
Possibility to automatically Activate the Smoke Evacuator VS 7001 Deltronix.

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PPC® Function

(Pulsed Polypectomy Cut/Coag)

Pulsed Cut and Coagulation for Polypectomy procedures and others.

This mode is a new way to use the CUT and COAG in ENDOSCOPIC techniques, increasing advantages in polypectomy procedures and others. In "PPC®" Function, radio frequency current applied to the polyp consists in cutting pulses interspersed with Coagulation current.

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COAGULATION MODE

6 Coagulation functions:

Desiccate, Spray, Fulgurate and 3 extra Pulsed Coagulation effects.

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COAGULACIÓN PULSADA

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TRIPOLAR® MODE

Tripolar® Mode is available ONLY in DELTRONIX® equipments. It allows simultaneous and independent use of both MONOPOLAR and BIPOLAR mode.

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COMPLATIBLE FOR CONECTION WITH ARGON PLASMA COAGULATOR

Allows connection to Argon Plasma Coagulator of any brand.

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BIPOLAR MODE

Three different BIPOLAR Modes:

PRECISE (Microbipolar), STANDARD (Bipolar), MACRO-BIPOLAR/BIPOLAR CUT (BCUT®). Independent footswitch for automatic activation of Bipolar mode functions.

AUTO STOP FUNCTIONS:

PRECISE (Microbipolar) STANDARD (Bipolar): When AUTO STOP function is chosen, Coagulation between Bipolar Forceps Tips is electronically monitored and the power selected is automatically canceled when tissue resistance reaches preset values (around 220 Ohms), even if the footswitch still pressed down, wich decreases significantly the risk of unwanted injuries in small blood vessels.

As higher the power is, the best and fastest is the coagulation..

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